# RECARO Sport HERO Reha

## Group I – III

(DE)	Montage- und Gebrauchsanleitung für Reha-Version (Ergänzung) .....	3
(GB)	Instructions for assembly and use of the Reha version (supplement) .....	5
(FR)	Montage et mode d'emploi pour la version Reha (complément).....	7
(ES)	Instrucciones de montaje y uso para la versión Reha (complemento).....	9
(IT)	Istruzioni di montaggio e di utilizzo per la versione Reha (aggiunta).....	11
(NL)	Montage- en gebruiksinstructies voor Reha-versie (aanvulling) .....	13
(NO)	Monterings- og bruksanvisning for Reha-versjonen (tillegg) .....	15
(SE)	Monterings- och bruksanvisning för Reha-Version (komplettering).....	17
(DK)	Monterings- og brugsvejledning for Reha-Version (tilføjelse) .....	19
(FI)	Reha-version asennus- ja käyttöohjeet (täydennys) .....	21
(PL)	Instrukcja montażu i użytkowania fotelika w wersji rehabilitacyjnej Reha (Uzupełnienie) .....	23
(GR)	Οδηγίες Συναρμολόγησης και Χρήσης για την Έκδοση Reha (Συμπλήρωμα) .....	25

# RECARO Sport HERO Reha

1a

1b

2a

2b

3a

3b

4a

4b

5a

5b

6a

6b

7a

7b

8a

8b

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## RECARO Sport HERO Reha – Instructions for assembly and use of the Reha version

### Dear RECARO Sport plus Reha user,

To make handling of your new children car seat safe, practical, and comfortable for both you and your child, please read these instructions and the instructions enclosed to the seat before use.

This manual is a supplement to the original RECARO assembly and usage instructions. Some aspects in handling the car seat have changed, due to the modification into a Reha version. These changed aspects are described in this supplementary user manual.

Should you have further questions or problems, please contact your specialist dealer or us directly.

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## General instructions

### Please note:

**The 5-point safety belt integrated in the seat is intended for the use in ECE group I (up to 18 kg / 39.7 lb) only. This requires that the seat must be secured with the vehicle's 3-point belt before.**

RECARO Sport HERO Reha has successfully passed an overload test at Dekra (Dresden) on the basis of the requirements of ECE R 44/04. Thus, the RECARO Sport HERO Reha is approved for the use as "restraint system for special needs ("S")".

### For use in ECE group II-III (18 kg/39.7 lb up to 36 kg/79.4 lb):

During the course of this test it was verified that the 5-point belt for children with a body weight of over 18 kg (39.7 lb) can be used as an additional positioning belt. In this case, ALWAYS secure your child using the vehicle's 3-point safety belt and observe the notes on correct installation contained in the original RECARO Instructions for Installation and Use.

The 5-point positioning belt integrated into the child seat is NOT a safety-relevant item. In this case the child is restrained by the vehicle's 3-point safety belt. The 5-point belt is only intended for use as a positioning belt to provide additional support for children with a body weight of between 18 and 36 kg (39.7 and 79.4 lb).

To enable your child to use the RECARO Sport HERO Reha for the longest possible time, the 5-point belt system, including the guide rail, can be removed. When removing the 5-point belt system please observe the notes contained in the original RECARO Instructions for Installation and Use. If it is also intended to use a 5-point positioning belt for larger children, we recommend the use of our belt extension (item code.: 824)

## Purpose / indication

The children car seat RECARO Sport HERO Reha is designed for the transport of children with special needs in vehicles. The clinical pictures are amongst others:

- cerebral palsy
- hypotony syndrome of unspecified genesis
- muscular dystrophy
- different syndrome diseases

The car seat is characterized by its shell design and pad system, which provide for additional seating safety, stabilization of head and pelvis, and support / control of the feet.

Thus, the RECARO Sport HERO Reha meets the needs of disabled children very well.

## Application risks / contraindications

No application risks / contraindications are known.

## 5-point belt

For more information on the use of belts please consult the RECARO user manual provided with the seat.

### Adjusting the belt length:

Slightly lift the seat cover and release the fabric flap by opening the hook and loop. Now you can see the central belt adjuster in the seat shell. To change the belt length, press the lever down and adjust the belt by either pulling the strap coming out of the front of the seat (shortening) or by pulling out both shoulder belts (lengthening).

### Belt buckle:

Press the red dent of the buckle to open. Separate the two upper belt fasteners. To close the belt, bring the two upper belt fasteners together, insert them into the buckle from above and push together until they lock in place with an audible „Click“.

**Fig. 1a:** Adjusting the belt length

**Fig. 1b:** Opening the belt buckle („Click“)

## Headrest

Pull the covers as tight as possible over the side parts of the headrest and attach them to the screws at the top.

**Fig. 2a:** Pulling headrest over side part

**Fig. 2b:** Attachment to screw

## Lateral trunk support

Attach the lateral trunk support by placing it on the seat. Guide both hook and loop strips behind the back cover and hook and loop-fasten them behind the cover.

The cover can be removed and washed in the machine at up to 30° C / 86° F.

**Fig. 3a:** Positioning on the seat

**Fig. 3b:** Hook and loop fastening

## Lateral seat support

The RECARO Sport HERO Reha is fitted with a seat reducer. The lateral seat supports are fitted to this seat reducer. To fit them, pull the lateral seat supports over each of the side sections of the seat reducer. Then hook and loop-fasten the lateral seat supports to the outside of the seat reducer.

The cover can be removed and machine-washed at up to 30°C/86°F.

**Fig. 4a:** Pulling the lateral seat supports over the seat reducer

**Fig. 4b:** Hook and loop-fastening

## Footrest adapter (accessory)

item code: 866

Before assembly, clean the plastic surface on the underside of the seat with commercially available cleaning agents (e.g. water and rinsing agent) until it is completely free from grease.

Now remove the hook and loop strips (fleece) from the adapter plate and attach them to the underside of the seat. The hook and loop strips (hooks) remain on the adapter plate.

Position the seat flush in a 90° angle at the front of the adapter (handle of the adapter facing forwards) and tilt the seat over to hook and loop it to the adapter.

**Footrest:** The footrest is attached to the adapter.

**Fig. 5a:** Attaching hook and loop straps to the seat

**Fig. 5b:** Attaching seat to adapter

## Footrest, short 12 cm / 4.7" (accessory)

item code: 806 .1

Push the footrest tubes into the front of the adapter. To do this, depress the catch springs on the footrest. Always secure the footrest against falling out using the knurled nuts on the left tube of the footrest adapter. Multiple holes in the footrest tubes allow the footrest to be adjusted to match the depth of the vehicle's seat.

To stabilize the footrest on the vehicle floor adjust the height of the support bars by screwing them in or out **until they have contact with the vehicle**. Then lock the support bars with the hand knobs to prevent them unscrewing accidentally. Screw the hand knobs upwards as far as possible and tighten them fast.

**Lower leg length:**

The lower leg length is adjusted by changing the height of the footrest. Loosen both outer screws. Retighten them after each adjustment.

**Foot angle:**

Loosen the four screws left and right on the underside of the footrest to adjust the desired foot angle with the rails. Then retighten the screws.

**Fig. 6a:** Inserting the support

**Fig. 6b:** Adjusting height to vehicle

## Turning plate with footrest adapter (accessory)

item code: 820

Please note our separate instructions for mounting! Download: [www.thomashilfen.com](http://www.thomashilfen.com)

## Seatfix connection (for turning plate) (accessory)

item code: 818

Please note our separate instructions for mounting! Download: [www.thomashilfen.com](http://www.thomashilfen.com)

## Stabilising bow, rear (for turning plate) (accessory)

item code: 819

Please note our separate instructions for mounting! Download: [www.thomashilfen.com](http://www.thomashilfen.com)

## Stand, front (for turning plate) (accessory)

item code: 817

Please note our separate instructions for mounting! Download: [www.thomashilfen.com](http://www.thomashilfen.com)

## Seat wedge, bottom (accessory)

item code: 810

The seat wedge is positioned centrally below the car seat. The high edge is at the front to achieve an optimum rest position of the child.

The cover can be removed (zipper) and washed in the machine at up to 30° C / 86° F.

**Please note:**

Function of the turning plate is considerably reduced if being used in combination with the seat wedge. We therefore do not recommend using seat wedge and turning plate at the same time.

**Fig. 7a:** Positioning seat on seat wedge

**Fig. 7b:** Final position

## Table (accessory)

item code: 811

Mount the side parts of the table first, by pulling them over the round corners of the seat. The side with the larger fabric must face outwards. Attach the fabric to the wheels for the seat adjustment using the provided holes in the cover of the side parts. Then tuck the downwards overlying fabric below the seat cover.

Now fasten the table to the hook and loop straps of the side parts. Guide the belt around the back of the seat and fasten it to the hook and loop (fleece) at the side of the table.

**Note:**

For reasons of space the crotch pad should be removed when using the table.

The cover can be removed and washed in the machine at up to 30° C / 86° F.

**Fig. 8a:** Mounting the side parts

**Fig. 8b:** Attaching the table

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Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG